



# Oxivir<sup>®</sup> Tb Wipes

## Ready to use disinfectant wipes

Virucidal • Bactericidal • Fungicidal • Tuberculocidal

### How to Use

These usage directions are intended for use for daily (routine) or discharge (terminal) cleaning and disinfection protocols. Consult specific protocols for directions beyond the usage of this product.

For heavily soiled areas, pre-cleaning is necessary.



1 Check the expiration date on the Oxivir<sup>®</sup> Tb Wipes canister. Discard if expired and get another canister.



2 Pull the lid off of the canister, and then open the pop-top on the lid.



3 Remove the seal on the canister top.



4 Find the first wipe in the center of the roll, and pull it through the opening in the center of the lid.



5 Place the lid back on the canister and press down to affix the lid to the canister.



6 Remove a wipe from the canister. Do not fold the wipe.



7 Clean and disinfect the hard, nonporous surfaces or equipment surfaces. Oxivir<sup>®</sup> Tb does both in one application.

### When To Change To A New Wipe:



A When the wipe does not deposit liquid on the surface.



B When the wipe is visibly soiled.



C When you are moving areas from one room to another



8 1 minute to achieve bactericidal and virucidal disinfection.



9 Dispose of the wipe(s) in accordance with appropriate protocol. **DO NOT FLUSH WIPES DOWN TOILET.**



10 Close lid of canister securely when finished so wipes will not dry out.

Always wear personal protective equipment.